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2-Function Walker

Wersja/Version: DUAL

Typ/Type: VCBP0031

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English version on next page

Table of contents

Introduction	16
Indications.....	17
Contraindications	17
Product description.....	17
Usage	17
Height adjustment.....	19
Moving by walker with step function	19
Moving with blocked step function.....	19
Safety instructions	19
Technical parameters	21
Service and repair.....	21
Care and maintenance	21
Cleaning.....	21
Disinfection.....	22
Product disposal	22
Maintenance plan.....	22
Symbols	23
Contact details	24

Introduction

Thank you for choosing our product. We are deeply convinced that it will meet your expectations. Please read this document before first use. This manual is passed in order to provide the necessary information for the safe use of the device. The correct application of the recommendations will contribute significantly to extend the durability and aesthetics of the product. We want to emphasise that constantly we are trying to improve the quality of our products, which may result in minor changes that are not included in the manual. Please ensure that you complete the attached warranty form and return it to your distributor so that we can provide you with the best possible service.

ATTENTION

Always follow the basic precautions according to the information given below regarding safety when using the product. Read the entire manual carefully before use. Keep this manual for future reference.

WARNING!

Improper use can lead to death, serious injury, unsafe conditions or product malfunction.

ATTENTION!

Improper use may lead to personal injury and/or damage to the product.

SUGGESTIONS!

Follow the instructions below to keep your product in good working order.

If the product is used by another person, you must provide them with copies of these instructions for use.

These instructions for use relate to the design and specifications of the product at the time of publication. Due to design changes, some illustrations and photos in the manual may not correspond to the product you have purchased. We reserve the right to make design changes.

The manufacturer disclaims all liability for personal injury or property damage that may result from improper or unsafe use of this product.

Indications

Walking aid designed for disabled people for those with locomotor disorders. Ideal for rehabilitation process and physiotherapy (verticalisation, lower limb relief).

Contraindications

Contraindications to the upright position, injuries and dysfunctions of upper limbs preventing the patient from supporting themselves.

Product description

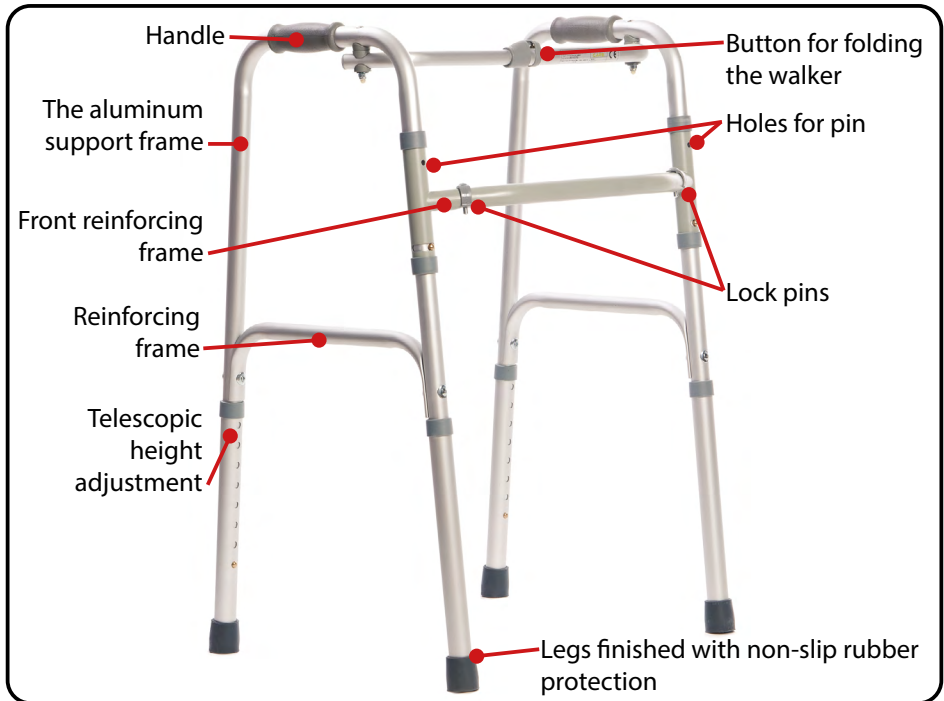
Figure 1.

Usage

- The product is packaged in a form to be assembled.
- After unpacking the package, check if it contains all parts in accordance with the list of elements set out above (Figure 1).
- Ensure that all components have no visible damage. If any doubts, please contact the distributor.
- In order to unfold the walker, should lean its sides maximally outside till the moment of latching the blockade. (Figure 3.).
- For easy storage and transport, the walker can be folded.
- To do that, should put lock pins moving function on the reinforcing frame.
- Than push the folding button placed on the frame and fold side frames.

Figure 2.

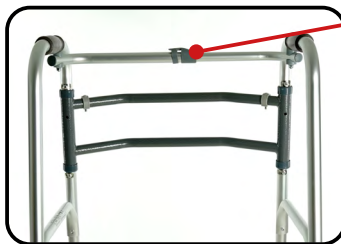
Figure 3.

Folding button

WARNING!

Special care must be taken when folding the rollator as there is a risk of jamming fingers or hands.

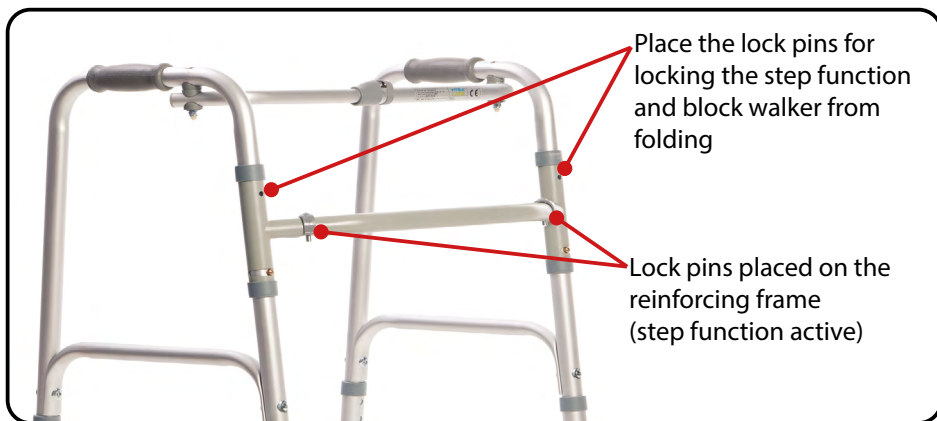
Figure 4.

Figure 5.

- On the reinforcing frame are located lock pins stepping function (Figure 5.). They are located in holes protecting to lose (Figure 4.).
- In order to block step function, should remove lock pins from the reinforcing frame and place it in position to block walk function (Figure 4.).
- Locked step function prevents walking movement and folding of the walker frame.
- For easy storage and transport, the walker can be folded.
- To do that, should put lock pins step function on the reinforcing frame.
- Than push the folding button placed on the frame and fold side frames.
- Release folding button. Walker should be folded to position shown on the (Figure 2.).
- Walker can be moved, held by side frames.

Height adjustment

- Before using a walker, should adjusted to the user's height by telescopically adjustable feet length.
- To do this, press the height adjustment pin of the legs and depending on the needs reducing the height of the legs or extend the leg (Figure 4.).
- It is important to the length of all legs was the same (handles must be parallel to the ground).
- Adjust the height of the walker so that the handles are at a height that provides comfort and a sense of security for the user.

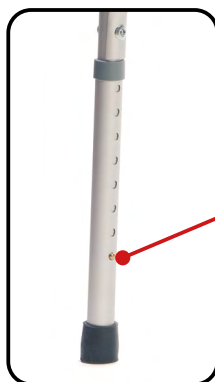
Figure 6.

Walker height
adjustment pin

Moving by walker with step function

- The step function is enabled when the pins lock are located in the reinforcing frame.
- Please stand near by walker, holding the handle with both hands.
- In order to use the device, interchangeably lift up the walker's lateral parts and move them forward and then put feet.

Moving with blocked step function

- In order to get walker constant function should remove the lock pin moving function from reinforcing frame and put it in lock position walk function and walker folding (Figure 4.).
- In order to use the walker constant function, lift it up, move forward and approach it.
- In order to get the moving function or fold the walker, put lock pin in the reinforcing frame.

Safety instructions

WARNING!

Any "serious incident" related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

ATTENTION!

The product is intended for use by people with disabilities and movement disorders. If you are not sure whether your health condition allows you to use the product, consult a healthcare professional.

WARNING!

Do not modify the product in any way; this will void the warranty.

**WARNING!**

Exceeding the maximum user weight will void the warranty. The manufacturer is not responsible for injury or damage caused by failure to comply with the maximum user weight.

**WARNING!**

Before using the walker, refer to this manual and keep it for future reference.

- Use of the product on the icy, wet, greasy, strongly gritty surfaces is unacceptable - it creates a risk of falling.
- The walker should be used on even surfaces at an incline less than 5% (2,86°).
- Before each use of this product:
 - Perform visual inspection of all structural components of the product,
 - Carefully check the stability and condition of the support frame.
- Do not use the walker in the event of any visible or perceptible damage (e.g. cracks in the frame). It may cause an accident.
- The walker is designed for users with a maximum weight of 110 kg.
- The walker is design to support one person, climbing and standing on the walker is strictly forbidden.
- Do not leave the walker close to open sources of fire.
- Be aware that the surface temperature of the upholstery and the frame parts may be increased after exposure to the sun or other heat source. Furthermore, with low temperatures there is a danger of overcooling metal parts of walker. It may cause burns or frostbite parts of the body.
- It is unacceptable to use the walker after discovering any visible or detectable damages (e.g. broken frame) as it may lead to accident.
- All the repairs needs to be performed by a skilled personnel.
- Do not exceed the maximum height.
- Keep walker with both hands simultaneously, in the direction of movement.
- Make sure that the lock used to folding and unfolding the walker is working properly.
- Manufacturer prohibits making any walker modifications on their own. It might cause a fall or serious injuries. The manufacturer will not take responsibility for such actions.
- Device is equipped with brakes located on the rear wheels.
- There is a risk of jamming fingers when folding the walker frame.

**WARNING!**

Maximum user weight 110 kg.

Technical parameters

Option	2FSS	2FSD
Maximum user weight	110 kg	
Walker hight	795-970 mm	820-990 mm
Walker width	530 mm	540 mm
Walker length	460 mm	490 mm
Walker weight	2,5 kg	
Range of height levels	8	
Adjustment level	25 mm	
Maximum safe inclination	5% (2,86°)	
Materials	Aluminium, poliuretan, guma	
Description of the environment in which the product will be used	  	
Permitted temperature	Od +5°C do +40°C	
The humidity for the storage and use	Od 30% do 70%	

Service and repair

The review should be carried out regularly, it should be preceded by thorough cleaning the product (so as to reveal any hidden damage). If repairs are needed or if you have any questions about the product, please contact your distributor.

Care and maintenance

Cleaning

- Clean the frame with a damp cloth, clean the seat with damp cloth with mild detergent.
- Remove all soiling from the movable components of the walker (wheels, clamping screws, locks).
- In order to maintain the walker parameters, it is very important to keep all elements clean and stored properly. Storing the walker in damp places (baths, laundry rooms, etc.) or outdoors, combined with lack of hygiene will quickly degrade the performance and aesthetic of the walker.
- Do not expose the walker on to extreme weather conditions (rain, snow very strong sun), because certain parts can change color.

It is not allowed to use:

- solvents
- toilet detergents
- sharp brushes and hard objects
- detergents that contain chlorine
- corrosive detergents

Do not clean the walker with pressure or steam aggregates!

Disinfection

If the product is used by several people (e.g. in a nursing home), commercially available disinfectants should be used. Before disinfecting, clean the upholstery and handles. Only tested and approved disinfectants may be used for disinfection.

Information on recognized and approved disinfectants and disinfection methods is available from your national health authority.

**WARNING!**

The use of disinfectants can damage the outer surfaces, which can reduce the long-term functionality of components. Follow the instructions provided by manufacturer of the chemical product.


Product disposal

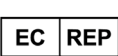The product must be disposed of in accordance with applicable local and national regulations. For information on the proper disposal of packaging, metal components, plastic components, neoprene, silicone and polyurethane materials, please contact your local waste disposal company or authorized distributor.

Maintenance plan

Frequency	Object of control	Comments
Before first installation	Checking the technical condition and all the elements within the packaging	Check if any elements have any visible mechanical damage
Every two weeks	Screw and nut control regarding their correct fixing	Make sure if all the screws and top are fixed tightly
Every two months	Cleanliness and general condition	In case of mechanical damage contact the producer immediately

Symbols

	Medical device
	Date of manufacture (printed on product)
	Manufacturer
	Distributor
	Authorized representative in the European Community
	Conformité Européene (compliance with the European Community)
	Catalog number (to re-order)
	Serial number
	Lot code
	Unique Device Identifier
	Refer to the instructions for use.
	Warning
	Possibility of jamming fingers
	Permitted temperature

	Protect from humidity
	For indoor/outdoor use
	Keep away from sunlight
	Maximum safe inclination
	The product is designed to be transported by car
	The product is not intended to be used as a seat for transport in a motor vehicle
	Maximum user weight

Contact details

Manufacturer

mdh Sp. z o.o.

Address: ul. Maratońska 104, 94-007 Łódź, Poland

tel. +48 42 674 83 84, fax. +48 42 636 52 21

www.mdh.pl www.viteacare.com

We reserve the right to make technical changes due to technical progress.